

3M CAVILON

No Sting Barrier Film



Procure No. 53712952 – 1ml wipe 53696306 – 28ml bottle

DESCRIPTION

3M Cavilon No Sting Barrier Film is a polymeric solution which forms a uniform film when applied to the skin. The product is dispersed in a unique non-cytotoxic, non-stinging solvent, which dries rapidly. Cavilon No Sting Barrier Film helps to protect intact or damaged skin from irritation caused by urine and/or fecal incontinence, digestive juices, wound drainage, adhesives, and friction. The film is colourless, transparent, and possesses good oxygen and moisture vapour permeability.

INDICATIONS

Cavilon No Sting Barrier Film is a liquid intended for use as a film-forming product, that upon application to intact or damaged skin forms a long lasting waterproof barrier, which acts as a protective interface between the skin and bodily wastes, fluids, adhesive products, and friction. It is intended as a primary barrier against irritation from body fluids. The product may be used on adults, children, and infants over one month of age.

CONTRAINDICATIONS

Not to be used:

- As the only covering in situations that require dressing protection from bacterial contamination/penetration, e.g. intravenous therapy catheter sites and full or partial thickness wounds
- On infected areas of the skin

Warnings:

- Cavilon No Sting Barrier Film is extremely flammable until it has completely dried on the skin
- It should only be applied when no ignition sources or heat-producing devices are in use – avoid using around flames
- Applicators and wipes are individually packaged for single use only, reuse could result in increased risk of infection or inadequate product performance

PRECAUTIONS

- Should redness or other signs of irritation appear, or if redness or signs of irritation persist, consult a physician
- Use of other barrier products, ointments, creams or lotions may significantly reduce its effectiveness

DIRECTIONS FOR USE

Instructions for use in a Surgical Setting

Discussion of use in this setting should occur during the time out period for verification of surgical procedure and site.

1. Skin should be clean and dry prior to application
2. Using the foam applicator, apply a uniform coating of film over the entire area of concern – the spray is not recommended for use in a surgical setting
3. If an area is missed, reapply to that area only after the first application is dry (approximately 30 seconds)
4. If applied to an area with skin folds or other skin-to-skin contact, ensure these areas are separated to allow the film to thoroughly dry before returning to the normal position
5. When used to protect skin from adhesive trauma, allow to thoroughly dry before covering with dressings, tapes or other adhesive products.

General Instructions for Use

1. Skin should be clean and dry prior to application.
2. Foam applicator or wipe – apply a uniform coating of film over entire area of concern.
3. Spray – hold nozzle 10 – 15 cm from skin and apply a smooth, uniform coating of film over the entire area of concern.
4. If and area is missed, reapply to that area after the first application has dried (30 seconds). If caustic irritants are expected a second coating can be applied after the first coat has dried.
5. In areas of skin folds ensure skin-to-skin contact is separated then allow film to dry before returning to normal position.
6. When used under adhesive tapes, dressings or devices – allow film to dry thoroughly before applying. Re-application of the film is required each time the products are changed.
7. If desired, the film can be removed by using moist medical adhesive removers as directed. Clean and dry the involved area and reapply the film.

FREQUENCY OF CHANGE

- Re-application every 24 – 72 hours when used to protect against body fluids, feces or urine etc.
- Re-application every 12 – 24 hours in extreme cases e.g. constant diarrheal stooling