

BACTIGRAS

Antimicrobial Mesh

Procure No. 53701511



DESCRIPTION

Bactigras consists of a cotton leno-weave fabric, impregnated with Soft Paraffin, containing 0.5% Chlorhexidine. Chlorhexidine is an antimicrobial agent that is active against a wide range of Gram-positive and Gram-negative bacteria but inactive against spores, fungi and viruses.

INDICATIONS

Bactigras is used as a wound dressing for the prevention of infection in minor skin loss injuries and ulcerative wounds. It is not recommended for the treatment of existing wound infection unless used as an adjunct to systemic antibiotic therapy where appropriate.

CONTRA-INDICATIONS

- Bactigras should not be used in patients who have a known sensitivity to Chlorhexidine

DIRECTIONS FOR USE

1. Prior to the application of the dressing, the wound should be cleansed with Normal Saline.
2. A single layer of Bactigras is then applied and covered with a secondary dressing.

FREQUENCY OF CHANGE

- Frequency of dressing change will depend upon the nature of the wound
- Wounds at risk of infection should be changed daily

Adapted from Smith & Nephew Product Information 2012