


INADINE

PVP-I Non-Adherent Dressing

Procure No. 53716498

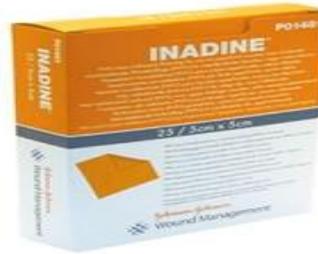

<p><u>DESCRIPTION</u></p> <p>Inadine PVP-I Non-Adherent Dressing is a topical wound dressing impregnated with an ointment containing 10% povidone iodine. The dressing also contains polyethylene glycol and purified water.</p>
<p><u>INDICATIONS</u></p> <ul style="list-style-type: none">• Designed to protect the wound even if infected• Management of ulcerative wounds and may also be used for the prevention of infection in minor burns and minor traumatic skin loss injuries• Provides a long lasting antiseptic effect, which helps manage infection by bacterial, protozoal and fungal organisms over a period of time• Suitable for use in adults and children• In heavily infected wounds, may also be used in conjunction with systemic antibiotics
<p><u>CONTRAINDICATIONS</u></p> <ul style="list-style-type: none">• Should not be used on patients who are sensitive to iodine or povidone-iodine
<p><u>PRECAUTIONS</u></p> <p>Should not be used in the following situations:</p> <ul style="list-style-type: none">• Where there is a known iodine hypersensitivity (allergy)• Before and after the use of radio-iodine (until permanent healing)• If you are being treated for kidney problems• In pregnant and breast feeding women• In cases of Dühring's herpetiform dermatitis (a specific rare skin disease) <p>Inadine dressing must be used under medical supervision:</p> <ul style="list-style-type: none">• In patients with thyroid diseases• In new born babies and infants to the age of 6 month as povidone iodine may be absorbed through unbroken skin• To treat deep ulcerative wounds, burns or large injuries
<p><u>DIRECTIONS FOR USE</u></p> <p>Site Preparation and Dressing Application</p> <ol style="list-style-type: none">1. Cleanse the wound by irrigating with sterile saline.2. Ensure skin surrounding the wound is dry.3. Remove the paper backing on each side of the dressing and apply the dressing to the wound base.4. Cover and secure with a secondary dressing of choice. <p>Dressing Change and Removal</p> <ol style="list-style-type: none">1. When the Inadine dressing turns white this indicates loss of antiseptic efficacy and therefore the dressing should be changed. It may be changed up to two times daily in the initial phase or with highly infected wounds or wounds producing large amounts of exudate.2. Removal may be helped by gentle stretching of the dressing at diagonally opposite corners.
<p><u>FREQUENCY OF CHANGE</u></p> <ul style="list-style-type: none">• Daily to every 3 days based on volume of exudate and release of iodine from the dressing

Adapted from Systagenix Product Insert 2011