

## INTERDRY AG

Polyester Textile with Silver

Procure No. 53694862



### **DESCRIPTION**

A non sterile skin protectant comprised of polyurethane coated polyester textile impregnated with an antimicrobial silver complex as the active component. The textile provides moisture transportation to keep skin dry while the antimicrobial in the textile reduces odour. Provides a protective environment for the skin and an effective protection against microbial contamination in the fabric. Provides antimicrobial action for up to 5 days. Interdry Ag reduces colonization of the following gram-positive and gram-negative bacteria as well as yeast: Staphylococcus aureus, Staphylococcus epidermidis, Pseudomonias aeruginosa, and Candida Albicans.

### **INDICATIONS**

- For moisture management of denuded skin
- For the prevention and management of intertrigo (where skin touches skin creating moisture and friction that causes redness, itching, burning, odour and/or maceration) in skin fold areas (i.e. groin, abdomen, axilla, toes, fingers, buttock)

### **CONTRA-INDICATIONS**

- Do not use with clients with known sensitivity or allergy to silver
- Do not use with creams, pastes, powders or ointments
- Do not use for wounds.

### **PRECAUTIONS**

- Do not use where there is a known sensitivity to silver
- Product is not to be washed

### **DIRECTIONS FOR USE**

1. Separate skin folds and cleanse. Pat dry.
2. Measure and cut length of Interdry Ag allowing a minimum of 5 cm (2") of free fabric which is to be exposed on either side of the treatment area.
3. Lay Interdry Ag into the base of the skin fold, smoothing it over the skin to remove any creases.
4. Gently place the skin fold together assuring that 5 cm of textile is exposed to air at each end.
5. Mark the exposed end of the textile with the date.
6. Separate the skin fold to assess the skin and placement of the textile daily or as indicated.

### **FREQUENCY OF CHANGE**

- On Days 2, 3, 4 & 5 remove the InterDry Ag and cleanse the skin and pat dry. Reposition same piece of InterDry Ag as above, unless it is soiled with urine, feces or excessive perspiration.
- On Day 5, discard cloth and apply new piece.

Adapted from Coloplast product information 2012