

## RESTORE CONTACT LAYER SILVER WITH TRIACT TECHNOLOGY

Antimicrobial Non-Adherent Dressing



Procure No. 53716458 4" x 5" size

<p><b><u>DESCRIPTION</u></b></p> <p>Restore Contact Layer Silver is a non-adhesive, non-occlusive, antimicrobial wound contact dressing, composed of a polyester mesh impregnated with a matrix comprising of hydrocolloid particles (carboxymethylcellulose), cohesion polymers, petrolatum and silver (2.25 mg/sq. inch). When in contact with wound exudates, the hydrocolloid particles combine with the matrix to form a lipido-colloidal gel, providing a moist environment that promotes healing. Being non-adhesive, removal of Restore Contact Layer Silver is virtually pain-free and helps minimize damage to newly formed surrounding skin. It is ideal for use on wounds with fragile surrounding skin. It was shown to be effective against bacteria most frequently associated with wound infection, Staphylococcus aureus, Streptococcus pyogenes, Pseudomonas aeruginosa (pyocyanic bacillus) and MRSA (strain ATCC 43300). The dressing sustains antibacterial activity for up to 7 days in in-vitro studies.</p>
<p><b><u>INDICATIONS</u></b></p> <p>The barrier functions of Restore Contact Layer Silver may help reduce infection in the following Low to Moderate exuding wounds:</p> <ul style="list-style-type: none"><li>• Second degree burns</li><li>• Pressure ulcers</li><li>• Venous stasis ulcers</li><li>• Diabetic ulcers</li><li>• Graft and donor sites</li></ul>
<p><b><u>CONTRAINDICATIONS</u></b></p> <ul style="list-style-type: none"><li>• Should not be used on individuals who are sensitive to silver or who have had an allergic reaction to the dressing or one of its components</li></ul>
<p><b><u>PRECAUTIONS</u></b></p> <ul style="list-style-type: none"><li>• Tends to stick to latex gloves – moisten latex gloves with sterile normal saline prior to use</li><li>• Concomitant use of other antimicrobial agent is not recommended</li><li>• Use in pregnant or breast-feeding women and newborns has not been studied</li><li>• Do not re-use the dressing</li><li>• Store dressing flat and at room temperature</li></ul>
<p><b><u>DIRECTIONS FOR USE</u></b></p> <ol style="list-style-type: none"><li>1. Cleanse wound using sterile saline solution.</li><li>2. Choose a dressing size that ensures the dressing will cover the entire wound.</li><li>3. Remove the protective tabs from the dressing.</li><li>4. Apply the dressing directly to the wound.</li><li>5. Cover with a secondary dressing and hold in place using a fixing bandage.</li></ol>
<p><b><u>FREQUENCY OF CHANGE</u></b></p> <ul style="list-style-type: none"><li>• Every 1 – 3 days depending on the wound and the healing progression</li></ul>

Adapted from Hollister Wound Care Product Insert 2011